



Access Your World

## Lopital Marina Shower Gurney / Bath Trolley

### Hydraulic Models Quick Use Instructions

#### Safety Tips:

- Read the user manual carefully before use (find it at [www.lopital.com](http://www.lopital.com)).
- Maximum load: 396 lbs.
- Make sure the brake is on during patient transfers.
- Check shower bed before each use: mattress intact, no bent or broken parts, and all moving elements function properly.

#### Use

- Make sure brakes are on.
- Press 'Up' to activate.
- Position stretcher appropriately, using 'Up' & 'Down' pedals.
- Fold side bar down.\*
- Move user to/from the Marina.
- Raise the bed base slightly. Close side bar.\*
- Disengage brake to relocate shower bed.

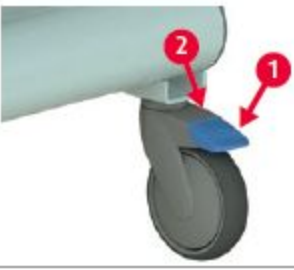

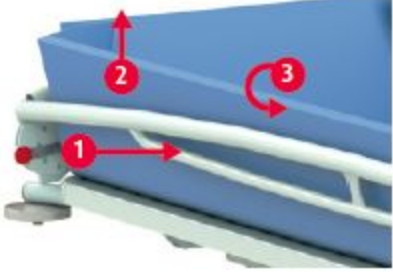
\* Use gentle pressure to avoid bending side-release tabs.



Subject to change without further notice



## Hydraulic Models Quick Use Instructions

<p><b>Wheel blocking</b></p>  <p>1. Wheel blocking. 2. Unlocking.</p>	<p><b>Hydraulic control</b></p>  <p>Upwards Downwards</p> <p>lopital<sup>®</sup> Marina</p>
<p><b>Side-bars</b></p>  <p>1. Unlock. 2. Tilt. 3. Turn. When closing, the bar locks automatically.</p>	<p><b>Cleaning and maintenance</b></p> <ul style="list-style-type: none"><li>• Clean after every use.</li><li>• Use water &amp; regular household cleaners.</li><li>• Do not use bleach or abrasive cleaners.</li><li>• Check wheels for hairs, debris, &amp; soap residue.</li></ul>

Call us at 800 577-1555 for support or assistance.

Subject to change without further notice

[www.max-ability.com](http://www.max-ability.com) 800-577-1555 Fax: 707-575-3856 email: [Info@max-ability.com](mailto:Info@max-ability.com)

rev. Nov. 2015